



Fitting Eyezen[™]+ Lenses

Eyezen+ lenses require the following standard measurements:

- Monocular Distance PD
- Fitting Height: Simply measure from center pupil to the lowest point of the lens to acquire fitting height. Recommended minimum fitting height is **15mm**.

Eyezen+ lenses with embedded Smart Blue Filter[™] feature are also recommended with Crizal[®] No-Glare treatment for the most comprehensive eye protection.

The following laser engravings located on the nasal side of each lens will identify the **Eyezen+** design received.

Eyezen+ 0



Eyezen+ 1




Eyezen+ 2



Eyezen+ 3



Visit EyezenPro.com/resources for additional information and materials.

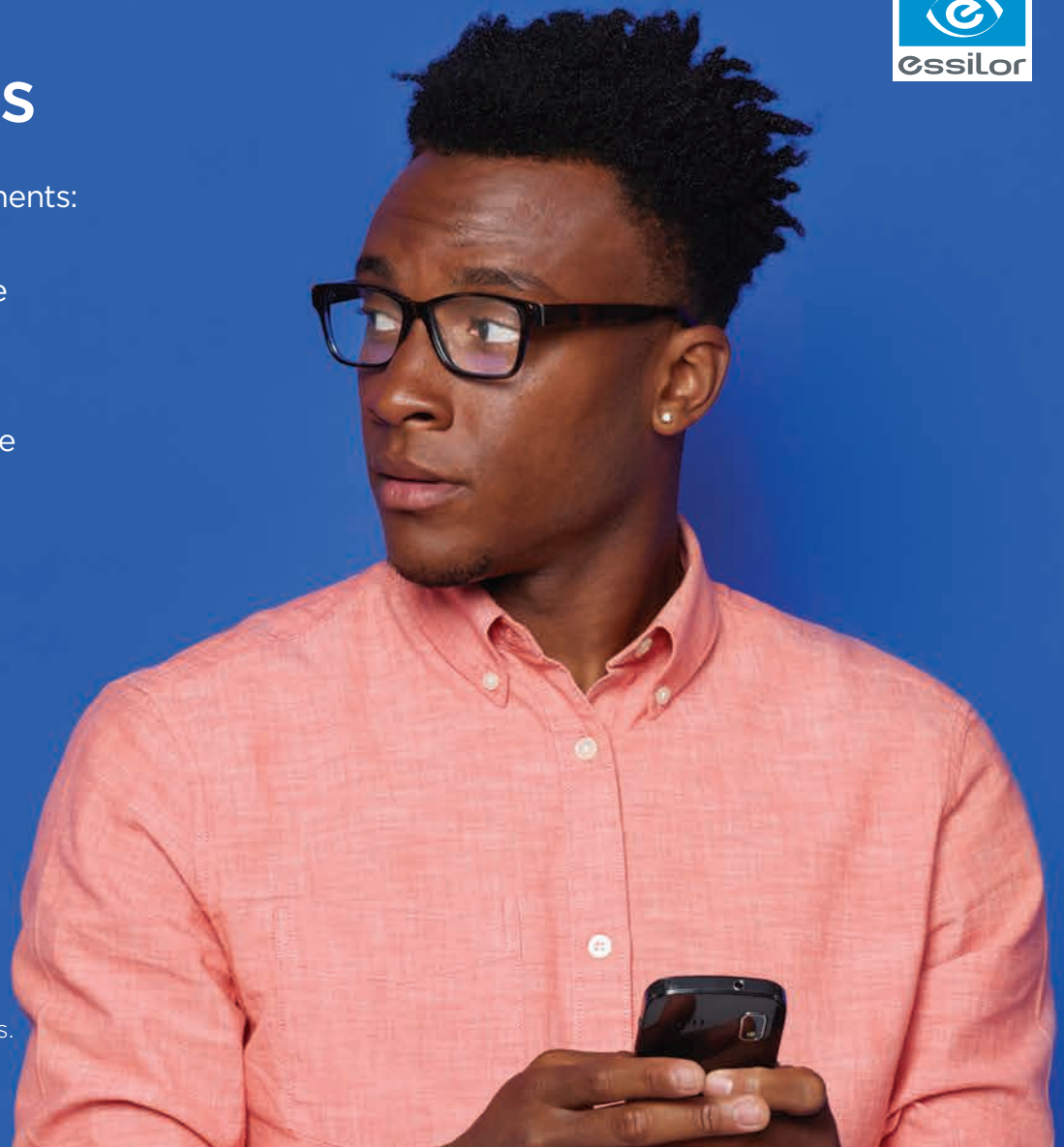

Embedded **Harmful Blue Light*** Protection.

*Eyezen+ lenses block at least 20% of Harmful Blue Light, which are the high energy waves found between 415 and 455nm (Blue-Violet light) and are suspected to be the most toxic to retinal cells.

Eyezen[™]+